smartbite / smartbite blackberry / smartbite bubble gum / smartbite cola / smartbite crystal clear / smartbite grey / smartbite speed

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Elastomeris Bissegistriermateriaal op Vinylpolyloxan-Basis, additionsvernetzend.

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Instructions for use

Elastomeric bite registration material on vinyl polysiloxane basis, addition-curing.

Thank you for the confidence you have shown in our company. You have made the right decision with the purchase of this bite registration material. Please be sure to read these instructions for use through carefully before starting to use the product! The bite registration materials are only intended for use in dental applications by correspondingly qualified professionals. The medical device must be employed responsibly and with due caution.

Product description:

All bite registration materials are produced from high-quality raw materials and suitable for a wide range of applications thanks to their different final hardnesses. The materials offer very high thixotropism, are easy to apply thanks to the automatic safety cartridge system, can be removed from the mouth with ease and display long-term dimensional stability.

Indication:

- smartbite / smartbite blackberry / smartbite bubble gum / smartbite cola / smartbite grey / smartbite speed: Production of long-lasting, reusable bite registrations; fixation of support and facebook registrations.
- smartbite crystal clear: Bite registration with required visual control; occlusal stamp in light-curing composite restorations; aid for production of temporary veneers; transfer matrix for fixation of brackets.

Bite registration material in 50 ml safety cartridges

Insert the cartridge in the 1:1 mixing gun, remove the end cap and dispense material until material is flowing evenly from both openings. Now place the mixing tip (single-use) over the openings (ensure that the colour code of the end cap and mixing tip are identical) and ensure that it is positioned correctly. To close it, turn the tip 90° clockwise. Operate the mixing gun again and dispense material in the desired amount. The used mixing tip remains on the cartridge and serves as a cap. Prior to the next use, remove the used mixing tip, check the opening for a plug that forms in isolated cases, clean if necessary and proceed as usual.

Disinfection:

The registrations can be disinfected with a 2% glutaraldehyde solution. After removing the impression from the mouth, rinse it under running water for 15 seconds. Then disinfect the impression by immersing it in the disinfectant thoroughly. Observing the disinfectant manufacturer's specifications is essential. Further information can be found at www.dgzmk.de, and in J. Prosth. Dent. 1999 May, 81(5), 621.

smartbite / smartbite blackberry / smartbite bubble gum / smartbite cola / smartbite crystal clear / smartbite grey / smartbite speed

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Rev. 04/2024 (2403_06/2024-03)

Working and safety instructions:

The products described above must only be used in accordance with their instructions for use. The user has sole liability for any other use which contravenes these instructions for use.

- Bite registration materials are not suitable for conventional impression-taking due to the strength of their final hardness.
- Close containers and tubes tightly immediately after use and do not interchange the lids and measuring spoons.
- Products marked "single use" are intended for a single use. To avoid cross-contamination, disinfect cartridges and mixing guns before the next use, e.g. with smartwipes disinfectant wipes.
- Solutions used prior to the registration, such as mouth rinses or retraction solutions, may interfere with the setting reaction and must be removed thoroughly.
- Contact between the material and certain types of gloves, e.g., latex gloves, should be avoided as the catalyst can be damaged. We recommend nitril / vinyl or PE-based gloves.
- Do not leave any bite registration material residues in the oral cavity.
- Avoid contact with clothing, as cross-linked silicones are chemically resistant and leave stains which cannot be removed.
- Do not use bite registration materials with polyether, polysulphides or condensation-curing silicones.
- To prevent possible incompatibilities, do not combine bite registration materials with A-silicones from other manufacturers.
- Storage: Store in a dry place at between 5°C and 27°C (41°F and 80,6°F).
- Shelf life: See date printed on package / label. Do not use after expiry date.
- Processing temperature: Room temperature between 18°C and 25°C (64,4°F to 77°F).

Technical data and product features

Product	Colour	Dosage (1:1)	Working time incl. mixing time*	Time in mouth	Total setting time	Hardness +/- 5	Linear dimensional change
smartbite	blue	automatic	30 sec.	1 min.	1 min. 30 sec.	Shore A85	0,1 %
smartbite blackberry	violet	automatic	30 sec.	1 min.	1 min. 30 sec.	Shore A85	0,1 %
smartbite bubble gum	pink	automatic	30 sec.	1 min.	1 min. 30 sec.	Shore A85	0,1 %
smartbite cola	brown	automatic	30 sec.	1 min.	1 min. 30 sec.	Shore A85	0,1 %
smartbite crystal clear	transparent	automatic	30 sec.	1 min.	1 min. 30 sec.	Shore A70	0,2 %
smartbite grey	grey	automatic	30 – 45 sec.	30 sec.	1 min. – 1 min. 15 sec.	Shore D43	0,1 %
smartbite speed	raspberry red	automatic	30 sec.	30 sec.	1 min.	Shore A85	0,1 %

* The times specified in the table above are based on 23°C and a relative humidity of 50 +/- 10%. As a general rule: High temperatures accelerate setting and lower temperatures delay it.

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Müller-Omicron GmbH & Co. KG